# Sådan gør du

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TOIMI NÄIN

# Instructions

For best results, read instructions carefully before use.

**Step 1. Brush your teeth** gently without toothpaste before each treatment.

## Step 2. Application

Application with your finger: Clean your hands, apply a drop of gel on the tip of the finger.

Then apply the gel on your teeth by gently rubbing the gel with your finger onto the teeth you wish to whiten. Then place the mouth tray on the row of teeth you want to whiten, both rows can be whitened at the same time.

## Alternative application

Application with finger: Wash your hands and place a drop of gel on to the tip of the finger. Apply by gently rubbing the gel with your finger onto the teeth you wish to whiten. Place the mouth tray on the row of teeth you have applied the gel to. Both top and bottom rows of teeth can be whitened at the same time.

**Step 3. Leave on your teeth for 15 minutes.**

**Step 4. Remove the tray** after 15 minutes. Rinse with water brush softly without toothpaste.

**Step 5. Repeat the treatment once a day** for 10 days or until you are satisfied with the results.

## Safety Instructions

If a large amount of gel is accidentally ingested, visit A&E immediately. If gel gets into your eyes, rinse with water immediately. Do not use if you suffer from irritated or inflamed gums, bleeding gums, oral infections, tooth decay, gum disease, if you are allergic to any of the ingredients or wear braces. You should allow your mouth and gums to heal before any teeth whitening treatment in order to refrain from irritating the wound / infection. If your oral infections persist you should visit the dentist. If irritation occurs, stop using the

## For the best Results

The number of treatments required for the desired results are different from person to person. Teeth are unique and will be whitened back to the natural white tooth shade of the individual. Repeat the treatment until desired results are achieved. We recommend that the treatment is completed. Results vary depending upon tooth discoloration and genetic conditions. Small white spots may appear on your teeth, don't be alarmed, this will soon disappear.

**Please Note:** fillings, veneers and other dental work will not be whitened as SimpleSmile® products are designed to whiten only the natural tooth enamel

## General Recommendations

- Avoid tea and coffee and coloured drinks for four hours after treatment.
- Teeth whitening is not recommended for persons under 18 years old.
- If you are pregnant or breastfeeding consult your doctor before starting treatment.
- Keep all teeth whitening products out of reach from children and animals.

## Storage

- Once opened, the expiry date of the syringe is 1 year.
- Best stored in refrigerator.
- Store the trays at room temperature and away from direct sunlight in order to retain the shape.

## Side Effects

There are currently no known side effects to SimpleSmile® teeth whitening and oral care products. SimpleSmile® products have been tried and tested and are produced in an FDA approved laboratory.

## More products for a white smile

For more information about SimpleSmile® products visit [www.simplesmile.co.uk](http://www.simplesmile.co.uk)

SimpleSmile® products are not tested on animals.

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