Sådan gør du

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TOIMI NÄIN

Instructions

For best results, read instructions carefully before use.

Step 1. Shape the mouth tray. Hold the tray by the tab and dip into water at a temperature of 80 °C for 2–3 seconds until it is malleable. Do not submerge the tab and be careful, keeping fingers out of the water at all times. Shake the tray to remove the remaining water, then carefully and quickly place the tray into the mouth on the upper or lower set of teeth (this will not burn you). Use your fingers and tongue to mould the tray to your teeth. Close your mouth and suck the air and moisture out, helping to provide a better impression of your teeth. Keep the tray in for 30 seconds.

If you are not satisfied, repeat the procedure with the same tray.

Once you are happy with the fit of the tray, rinse with cold water and remove the tab with a pair of scissors. If you are not happy with the result, there is an extra mouth tray with.

Step 2. Brush your teeth gently without toothpaste before each treatment.

Step 3. Using the syringe, apply a small line of gel to the mouth tray (approximately 0.5 ml) from one end to the other.

Step 4. Leave on your teeth for 15 minutes.

Step 5. Spit out and rinse with water. Brush your teeth **without toothpaste**.

Step 6. Repeat the treatment once a day for 10 days or until you are satisfied with the results. The product can be used for up to 20 days over a three-month period. Do not apply too much gel to the mouth tray, about 0.5 ml per treatment is sufficient. The syringe will then last for 10 treatments.

Scan the QR code to watch Video Tutorial on how to best shape the mouth trays.

Safety Instructions

If a large amount of gel is accidentally ingested, visit A&E immediately. If gel gets into your eyes, rinse with water immediately.

Do not use if you suffer from irritated or inflamed gums, bleeding gums, oral infections, tooth decay, gum disease, if you are allergic to any of the ingredients or wear braces. You should allow your mouth and gums to heal before any teeth whitening treatment in order to refrain from irritating the wound / infection. If your oral infections persist you should visit the dentist.

If irritation occurs, stop using the treatment.

For the best Results

The number of treatments required for the desired results are different from person to person. Teeth are unique and will be whitened back to the natural white tooth shade of the individual.

Repeat the treatment until desired results are achieved. We recommend that the treatment is completed. Results vary depending upon tooth discolouration and genetic conditions.

Small white spots may appear on your teeth, don't be alarmed, this will soon disappear.

Please Note: fillings, veneers and other dental work will not be whitened as SimpleSmile® products are designed to whiten only the natural tooth enamel.

General Recommendations

- Avoid tea and coffee and coloured drinks for four hours after treatment.
- Teeth whitening is not recommended for persons under 18 years old.
- If you are pregnant or breastfeeding consult your doctor before starting treatment.
- Keep all teeth whitening products out of reach from children and animals.

Storage

- Once opened, the expiry date of the syringe is 1 year.
- Best stored in refrigerator.
- Store the trays at room temperature and away from direct sunlight in order to retain the shape.

Side Effects

There are currently no known side effects to SimpleSmile® teeth whitening and oral care products. SimpleSmile® products have been tried and tested and are produced in an FDA approved laboratory.

More products for a white smile

For more information about SimpleSmile® products visit www.simplesmile.co.uk

SimpleSmile® products are not tested on animals.

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